

Endy NT2 | the micromotor with integrated light



ARE YOU LOOKING FOR AN EFFICIENT DEVICE WITH AN OPTIMUM CONTROL?

With Endy NT2, you have a tailored solution offering autonomy and comfort of use

a balanced compact motor with LED that can be autoclaved at 134°


The white LED under the contra-angle **illuminates the operating field without casting a shadow** or altering visibility.

a wide range of speeds with a single contra-angle

The Endy NT2 lets you set your file speed between 250 and 1200 revolutions per minute using the same contra-angle. **A single machine can suffice for widening all root canals.**

consistent torque and speed

The **speed** of the Endy NT2 **remains consistent regardless of the force with which the file is used in the root canal.** It never varies, even if the battery is less than 50% charged. The **torque is always controlled in order to preserve your files.**

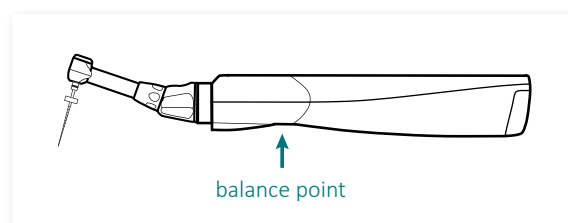

Unit drawing

YOUR PARTNER FOR OVER 25 YEARS

Established in 1993 and located in France in the Bordeaux region, **the company IONYX designs and manufactures specialized machines for endodontics.** Intended for general practitioners seeking to improve the quality of their treatment and their comfort, **the entire IONYX range is designed for daily and easy practice.**

Present everywhere in the world through its wide network of distributors, **IONYX is one of the most recognized experts in the field of dentistry.**



Endy NT2

micromotor with integrated light

TECHNICAL CHARACTERISTICS

rapid motor shutdown

As soon as the programmed torque is achieved, the motor is quickly shut down by means of an electronic lock, limiting the risk of breakage of the instrument.

interruptible slow reverse

This 1-second slow-reverse technology used by IONYX since 1996, lets you remove the instrument without sending dental debris back towards the apex.

long Li-ion battery life

The Endy NT2 lets you work on battery power for 1 hour with an average torque of 1.5 N.cm (40 root canal treatments on average reported).

quick charger

The device's quick charger fully recharges your battery in 2 hours. Your battery is protected against short circuits or any heating during the charging process, ensuring complete safety. A red indicator light indicates that the battery is in safety lockout.

10-memory settings


The Endy NT2 has a 10-memory settings for programming as many torques and speeds corresponding to each of the files used. Programming is easy and straightforward.



side view



front view

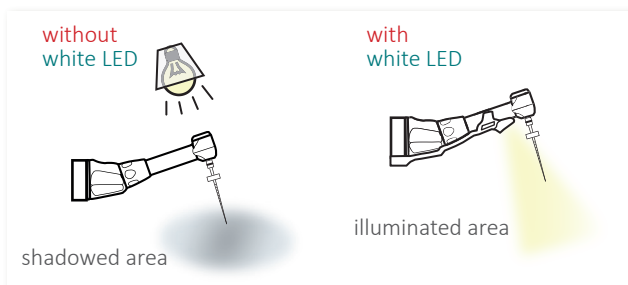

Autoclavable LED drawing

*The Endy NT2 electronic casing has a 2-years warranty.
The contra-angle, the battery and the accessories have a 1-year warranty.*

Learn about the other products in our **ionyx** range



Locapex 6
the new
apex locator
designed by ionyx

Endy 6200
micromotor
with apex locator



A REAL PROGRESS IN ENDODONTIC PRACTICE

ionyx

34 rue de Fleurenne
F-33290 BLANQUEFORT

T. +33 5 56 35 34 27
F. +33 5 56 95 16 20

ionyx@ionyx.eu
www.ionyx.eu

